BEDIENUNGSANLEITUNG
Operation manual
Istruzioni d'uso
Manual de instrucciones

ADE

Körperlängen-Messband
Body Measuring Tape
Altimetro a nastro
Cuerpo Cinta Métrica

MZ10017

MZ10017-120308-Rev002-UM-de-e-it-es

ADE GmbH & Co.
Hammer Steindamm 27 – 29

22089 Hamburg / GERMANY

☎ +49(40)432776-0
Fax +49(40)432776-10
E-mail: info@ade-hamburg.de

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Operation Manual

Packaging:

Body Measuring Tape
Fixing material (3 screws, 3 dowels)

Assembling:

- In order to find the correct fixing point put the whole unit on the ground at the wall.
- Pull the upper part up until the reading mark reaches `zero`.
- Use the fixing material to fix the upper part on the wall and re-check position.
- Tighten screws and make sure the whole equipment is now properly mounted.

Measuring:

- Place the person to be measured underneath the Body Measuring Tape with the back on the wall.
- Make sure the person's heels touch the wall and the head is in upright position.
- Pull the measuring part down to the head of the person.
- Reading mark is now indicating the length of the measured person.
- Take the measuring part back to starting position.

Cleaning:

- Clean the plastic parts with a damp cloth if necessary.
- Do not use any aggressive cleaning fluids.

Warranty

A two-year warranty from date of purchase will be done by repair or replacement once it is proved that the defects are caused by poor material or poor workmanship (please keep the receipt for presentation). All moveable parts such as batteries, cables, plug-in power supply units, rechargeable batteries etc. are excluded. Warranty does not apply to normal appearance of usage or sign of wear or any damages caused by accidents or misuse. Products which have been opened by unauthorized persons are automatically out of warranty. Customers abroad are requested to contact their local sales agent in the event of any warranty becoming applicable.

Technical data

- Measuring range: 0 – 2200 mm
- Graduation: 1 mm
- Dimensions: 140 x 130 x 35 mm
- Weight: abt. 180 g

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