

GEEMARC

CLEAR SOUND™

HEAR THE DIFFERENCE

AMPLICALL50

Portable Alert Unit



English

geemarc™
Telecom SA

CONTENTS

CONTENTS	1
INTRODUCTION	3
Unpacking the AMPLICALL™	50
	5
DESCRIPTION	6
General Description	6
INSTALLATION	8
Setting Up	8
Mounting	9
Range	10
PAIRING	11
Pairing Procedure	11

CONTENTS

Deregister the Pairing	13
AMPLICALL™50	16
Power On/Off Setting	16
Vibrate On/Off Setting	16
Volume Setting	16
AMPLICALL™50 Use	17
TROUBLESHOOTING	18
SAFETY INFORMATION	21
GUARANTEE	26
RECYCLING DIRECTIVES	29

INTRODUCTION

Congratulations on purchasing your Geemarc AMPLICALL™50. This is the portable unit that receives a transmitted signal from any number of transmitters e.g. AMPLICALL™30/40. The AMPLICALL™50 will flash and can emit an alarm sound and/or vibrate in order to alert you to help the person/baby in need.

The AMPLICALL™50 is a fantastic receiver able to link to many different types of transmitter within the range together with the obvious requirements of good,

INTRODUCTION

clear reception and easy setup and use.

Its is important that you read the instructions below in order to use your Geemarc AMPLICALL™50 its full potential. Keep this user guide in a safe place for future reference.

INTRODUCTION

Unpacking the AMPLICALL™50

When unpacking the AMPLICALL™50, you should find the following in the box:

- 1 AMPLICALL™50 Unit
- 3 AAA batteries (supplied)



For product support and help visit our website at www.geemarc.com


Telephone 01707 384438

Or fax 01707 832529

INTRODUCTION


General Description

Front Description

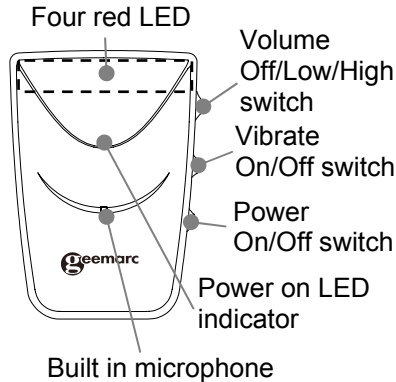

INTRODUCTION

Rear Description

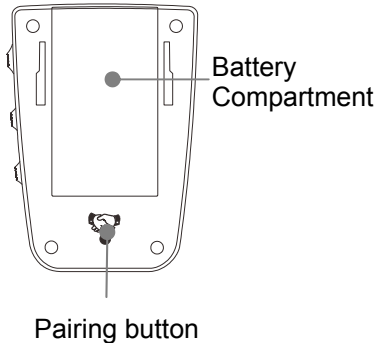

INTRODUCTION

Setting Up

For portable monitoring, remove the battery compartment cover found on the rear of the unit.

Insert 3 alkaline batteries (AAA size, 1.5V type, these are supplied) into the battery compartment.

Check the battery polarity diagrams in the battery compartment when inserting the batteries. Replace the battery cover – it will snap back into place.

Slide the Power On/Off switch to On to activate.

INTRODUCTION


When the LED indicator flashes

- Slowly-Normal
- Quickly-Low battery(replace battery)



Mounting

The AMPLICALL™ 50 can be carried around with you either by hand or using the supplied belt clip.

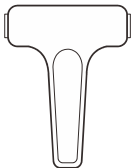

INTRODUCTION

Range

Your monitor has been carefully designed to operate at its best within a 30 metre range. This is suitable for the majority of household environments. The range and general quality of reception may vary depending on battery power and number of obstructions such as walls etc.

PAIRING

The Pairing Procedure

Press and hold the Pairing/Test button of the AMPLICALL™ 50 until the unit makes a beep sound. The 1st LED lights. This indicates you have entered pairing mode.

You can now pair the 1st outdoor unit/transmitter e.g. the AMPLICALL™ 30/40 to the 1st LED. Press and hold the Pairing/Test button of AC50 and the **ALARM** button of the outdoor unit/transmitter e.g. the AMPLICALL™ 30/40 at the same time until the middle LED lights up

PAIRING

and the unit makes a long beep tone to confirm pairing successfully.

If you would like to continue and pair the 2nd, 3rd or 4th outdoor unit/transmitter, when the 1st LED lights, press the Pairing/Test button again. The 1st LED will switch off and the 2nd LED will light up. You can now pair the 2nd outdoor unit by pressing and holding the Pairing/Test button of AC50 and the **ALARM** button of the outdoor unit/transmitter at the same time until the middle LED lights up and the unit makes a long

PAIRING

beep tone to confirm pairing successfully.

You can repeat the same procedure for the 3rd or 4th outdoor unit/transmitter.

Deregister the Pairing

Press and hold the Pairing/Test button for 5 seconds until all the LEDs light up. You have entered the deregister mode.

In order to deregister the 1st unit, press the pairing/testing button once, the 1st LED will flash (the other 2 LED will be lit), then press

PAIRING

and hold the pairing/testing button. The 1st LED turns off and beeps to confirm that 1st remote is now deregistered

If you would like to deregister the 2nd and 3rd unit but keep the 1st unit paired:, when the 1st LED flashes, press the pairing/testing button once to select which call/remote unit is to be deregistered, then press and hold the pairing/testing button until the 2nd or 3rd LED turns off and beeps to confirm that it is now deregistered.

To keep the 1st, 2nd & 3rd units paired, but deregister the 4th :

PAIRING

when the 1st LED flashes, press the pairing/testing button four times, all 3 LEDs will flash together, then press and hold the pairing/testing button until all 3 LEDs turn off together and beep to confirm that 4th unit is now deregistered.

USING THE AMPLICALL50

Power On/Off Setting

You can switch the power off or on using the Power Off/On switch.

Vibrate On/Off Setting

You can switch the vibrate setting off or on using the Shaking Off/On switch. When this setting is on the AMPLICALLTM50 will vibrate as a means of alarm.

Volume Setting

The beep alarm can be set to off, low volume or high volume using the Volume off/low/high setting.

USING THE AMPLICALL50

AMPLICALL™ 50 Use

When a paired transmitter detects a cry/voice/noise or the ALARM button is pressed, this receiver unit will flash and/or sound a beep and/or vibrate to notify you of the noise made and perhaps the need for assistance.

TROUBLESHOOTING

The Monitor will not switch on

- Make sure the batteries are inserted correctly and they have adequate power
- Ensure mains power is connected properly

The Receiver (AMPLICALL™50) is making a high pitched (feedback) noise

- Ensure the AMPLICALL™30/40 and the AMPLICALL™50 are about 10ft apart
- Switch the volume setting to low

TROUBLESHOOTING

The Receiver (AMPLICALL™50) is only alerting me intermittantly or not alerting me at all

- Ensure the units are paired correctly. Run through the pairing process again
- Ensure the AMPLICALL™30/40 and the AMPLICALL™50 power is switched on
- Ensure the AMPLICALL™50 volume is not switched off
- Make sure the batteries are inserted correctly and they have adequate power
- Ensure mains power is connected properly

TROUBLESHOOTING

- The units maybe too far apart, therefore out of range
- Set time delay to low and sound sensitivity to high

Sound quality has decreased

- Make sure the batteries are inserted correctly and they have adequate power

Power LED flashing quickly

- Replace the old batteries or use mains power

SAFETY INFORMATION

General

Do not open the unit (except to replace the batteries). Contact the helpline for all repairs.

One of the uses of the AMPLICALL™50 is to help monitor your child and is not a substitute for adult supervision.

This product is not a toy.

Your monitor transmits over public airwaves. Your conversation maybe heard by another product that operates on the same frequency.

SAFETY INFORMATION

Always keep electrically operated items and their cables out of reach of babies and younger children.

Ensure children cannot become tangled or trip over any wires.

Cleaning

Clean the unit with a soft cloth. Never use polishes or cleaning agents - they could damage the finish or the electrics inside. If connected, please disconnect from power supply before cleaning.

SAFETY INFORMATION

Environmental

Do not expose to direct sunlight.

Always ensure there is a free flow of air over the surfaces of the unit.

Do not place any part of your product in water and do not use it in damp or humid conditions e.g. bathrooms.

Do not expose your product to fire or other hazardous conditions.

SAFETY INFORMATION

Batteries

Removal and replacement of batteries should be carried out by an adult or under adult supervision.

Take special care to ensure batteries are inserted correctly, observing (+) and (-) polarity marks on battery and product.

Remove used batteries from the unit.

Do not use rechargeable batteries.

SAFETY INFORMATION

Do not mix old (used) and new batteries or batteries of different types

Do not put batteries into the household rubbish, pay attention to battery regulations regarding battery disposal.

Remove batteries if you will not be using the product for long periods.

Only use batteries of the same or equivalent type to those recommended.

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of one year. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at www.geemarc.com.

The guarantee does not cover accidents, negligence or breakages to any parts.

The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc

GUARANTEE

guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only

DECLARATION: Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and

GUARANTEE

other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5/EEC and in particular article 3 section 1a, 1b and section 3.

The declaration of conformity may be consulted at www.geemarc.com



RECYCLING DIRECTIVES

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin.

Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip.
- Deposit the product in an appropriate WEEE skip.

RECYCLING DIRECTIVES

- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.



For product support and help visit our website at www.geemarc.com

Telephone 01707 384438

Or fax 01707 832529



UGAmplicall50_En_v1.3

geemarc United Kingdom

5 Swallow Court
Swallowfields
Welwyn Garden City
Hertfordshire, AL7 1SB
For product support:
01707 384438

www.geemarc.com

geemarc France

Parc de l'Etoile
2, rue Galilée BP20061
59791 GRANDE SYNTHÉ CEDEX
TEL. SERVICE APRES VENTE:
03 28 58 75 99

www.geemarc.com

geemarc Deutschland

Siemensstrasse 10,
53121 Bonn
Deutschland
Tel: +49(0)228/ 74 87 09 0
Fax: +49(0)228/ 74 87 09 20

www.geemarc.com