

Function


Thank you very much for buying the machine.

When you see film, watch the play, have meeting, or have classes, the product can help you to hear the sound clearly and loudly. While in forest, if you want to hear very slight sound, it will be a good choice.

Specification

1. Max sound output: 130±5dB
2. Sound gain: ≥ 50dB
3. Total harmonic wave distortion: ≤ 5%
4. Frequency range: 300~4000Hz
5. Input noise: ≤ 30dB
6. Voltage: d.c. 1.5V
7. Current: ≤ 4mA

Instruction diagram

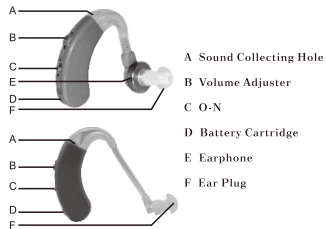

“N” means “Power On”
“O” means “Power Off”
“+” Positive electrode of Battery
“-” Negative electrode of Battery
“1~4” Volume grade, “1” Min,
“4” Max

Usage

Open the cartridge cover, put the battery in correct direction. Zinc Air Battery is best choice and has longer life.



1. Before use, make sure the power is turn off, or make the volume to “1”.



2. Put the ear hook on your ear, plug in the ear plug tightly. (Note: The unit is on “off” position. And volume is on “1”, otherwise the instant sound will make you uncomfortable).

Usage

3. Turn on the power and adjuster the sound volume to get clear sound.



4. If you hear the whistling sound, please check whether you have plugged in the ear plug tightly. If it can't be plugged tightly, please change the other size plug.



W: 22.1cm

H: 6.8cm

Maintenance

No sound. Check as follows:

1. If you turn on the power.
2. If you fit the battery in correct direction.
3. If the battery power has been used up.
4. If the earphone is damaged.
5. If the lead is broken.

The sound is very weak and vague.

1. Battery power isn't enough.
2. If something is wrong with earphone.

Use dry cloth and special detergent to clear unit body.

Note

1. It is forbidden to used by those patients with tympanitis or ear pyogenesis.

2. Don't use it at high volume for long time. Otherwise it will injure your ear. If you feel uncomfortable, please stop using.
3. Please maintain the products all the time, avoid ear plug and sound collecting hole being blocked by foreign objects. It will affect the effect of sound amplifier.
4. Avoid shocking ,falling the product. Don't keep or use it in environment of humid of high temperature.
5. Don't throw the scrapped battery as common rubbish,it should be handled by special equipment.

Note

6. Please take out the battery if don't use it for a long time. Otherwise the components will be rotted by time. rotten liquid from battery.
7. Don't use the product in potential explosive or dangerous environment.
8. Keep the unit away from the children. It will injure him or others, or damage the unit.

Company Address :
Mabis Healthcare UG
George Fahrback-Strasse 5
D-55129 Mainz
Germany

AMPLIFIER HEARING AID Operation Manual

MODEL NO.:AVA115



 **MABIS**
HEALTHCARE UG GEBMANT
Taking Care of your healthcare needs