SCOPE ONE

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Instruction manual

Mode d'emploi

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How to use the Otoscope :

- 1 / Turn on the device by pushing the power button down
 - 2 / Choose a speculum adapted to the ear of the patient, fix it on the head of the otoscope pushing the speculum and by turning it in an anti-clockwise direction.
 3. Introduce the instrument inside the ear canal of the patient and review via the observation window
- Caution: Do not examine the ear of the patient without disposable speculum

Maintenance

- Replacing the light bulb:
1. Turn the power off and unscrew the handle from the head
 2. Remove the old bulb by pulling it downwards
 3. Insert a new bulb and screw the handle to head
- Caution: Use only the LED Scope One bulb that is designed for this device
- Battery replacement:
1. Turn the power off and unscrew the bottom of the handle
 2. Remove the used batteries and insert fresh AA alkaline batteries
- Caution: Do not leave dead batteries too long into the unit as this may damage the device. Dispose of used batteries in accordance with the regulations in force

Cleaning and disinfection:

Clean the surface of the device frequently with 75% alcohol. Clean the magnifying glass with a dry towel.

Caution: Do not immerse the Device and keep out of any liquid

SCOPE ONE

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frakita

www.frakita.net

EC REP
Blue Start Company, 40 Thorncliffe Road,
Norwood Green, Southall, Middlesex, UB2 5RQ UK

LAFTAN TRADING INTERNATIONAL
Haji Pura Fateh Gerh Road – Sialkot - Pakistan