BF206-BT

Bluetooth[®] Body Composition Monitor

Intended Use

Medical Purpose

Dear customer,

Thank you for purchasing this high-quality OMRON Body Composition Monitor. The BF-206BT is intended to measure and display the following body composition:

- Body weight
- Body Fat (in %)
- Skeletal Muscle (in %)
- Visceral Fat (up to 30 levels)
- Body Mass Index (BMI)
- Body Age

In addition, the calculation of the Resting Metabolism Rate (in kcal) determines the daily calorie requirement and can act as a reference for weight reduction programme.

The unique OMRON 4-sensor technology provides one of the most accurate methods of body measurements.

Suitable Users

The BF206-BT is intended to measure the people matching the following conditions.

- Height: 100.0 to 199.5 cm
- Weight: 150 kg or less
- Age: All users*

* The age range is different according to the measurement parameter.

Please refer to "Setting Items" in the Technical Data Section.

Precautions for Use

Please read this instruction manual carefully before use and for further information on the individual functions.

Contents

Before using the unit

Notes on Safety	2
Information on Body Composition	3

Operating instructions

1. Know Your Unit	5
2. Inserting and Replacing the Batteries	5
3. Setting the Date and Time	6
4. Setting and Storing Personal Data	7
5. Taking a Measurement	9
6. Measuring Weight Only	12
7. Viewing the Last Measurement Results	13
8. Transferring the Data	14
9. About the Wireless Communications Function	16

Care and maintenance

10. How to Take Care of and Store the Unit	16
11. Error Displays	17
12. Troubleshooting	17
13. Technical Data	18

EN

Notes on Safety

Symbols and definitions are as follows:

 Danger:	Improper use may cause danger resulting in death or serious injury.
 Warning:	Improper use may result in possible death or serious injury.
 Caution:	Improper use may result in injury or property damage.

Danger:

- Never use this unit in combination with medical electronic devices such as:
 - (1) Medical electronic implants such as pacemakers.
 - (2) Electronic life support systems such as an artificial heart/lung.
 - (3) Portable electronic medical devices such as an electrocardiograph.

This unit could cause these devices to malfunction, posing a considerable health risk to users of these devices.

Warning:

- Do not begin a weight reduction or exercise program without consulting a doctor or healthcare provider first. Self-diagnosis could damage your health.
- Do not use the unit on slippery surfaces, such as a wet floor.
- Keep the unit out of the reach of young children.
- Do not jump onto the unit, or hop up and down on the unit.
- Do not use this unit when your body and/or feet are wet, such as after taking a bath.
- Stand on the unit bare-footed. Attempting to stand on it with socks on may cause you to slip and result in injury.
- Do not step on the edge or display area of the unit.
- People with disabilities, or who are physically frail, should always be assisted by another person when using this unit. Use a handrail or so when stepping on the unit.
- If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Consult a doctor immediately.

Caution:

- This unit is intended for home use only. It is not intended for professional use in hospitals or other medical facilities. This unit does not support the standards required for professional use.
- Do not disassemble, repair, or remodel the unit.
- Do not use mobile phones near the unit except for wireless communications.
- Do not use batteries not specified for this unit. Do not insert the batteries with the polarities in the wrong direction.
- Replace worn batteries with new ones immediately.
- Do not dispose of batteries in fire.
- If battery fluid should get on your skin or clothing, immediately rinse with plenty of clean water.

- Remove the batteries from this unit when you are not going to use it for a long period of time (approximately three months or more).
- Do not use different types of batteries together.
- Do not use new and worn batteries together.
- Always wash your feet before using the unit if you are suffering from a foot infection or other skin disease. Failure to do so could infect others. When the unit will be used by several people, wipe the unit with a damp cloth moistened with mild detergent after using it. Then wipe it dry.

General Advice:

- Do not place this unit on a cushioned floor surface such as on a carpet or a mat. Correct measurement may not be possible.
- Do not place this unit in highly humid environments and protect it from splashing water.
- Do not place this unit near heat sources or below air conditioners and avoid direct sunlight.
- Do not step on the display or buttons of this unit.
- Do not use this unit for purposes other than described in this manual.
- As this unit is a precision instrument, do not drop, vibrate, or apply strong shock.
- Disposal of used batteries should be carried out in accordance with the national regulations for the disposal of batteries.
- Do not wash the unit with water.
- Do not wipe the unit with benzene, gasoline, paint thinner, alcohol, or other volatile solvents.
- Do not place this unit where it will be exposed to chemicals or corrosive vapors.

Read and follow the "Important information regarding Electro Magnetic Compatibility (EMC)" in the Technical Data Section.

Information on Body Composition

Principle of body composition calculation

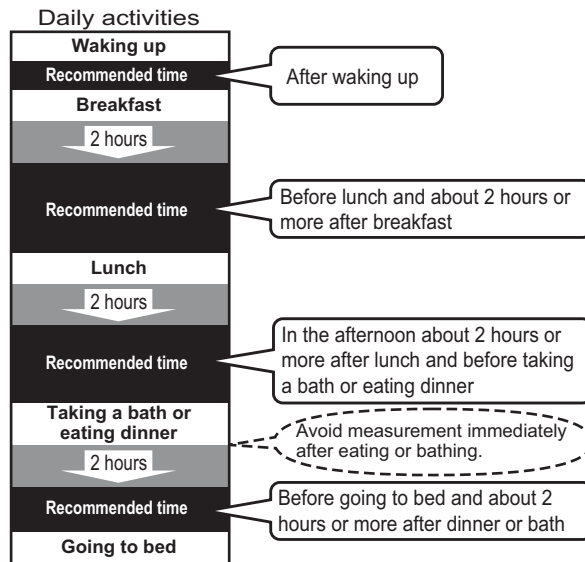
Body fat has low electric conductivity

The BF206-BT measures the body fat percentage by the Bioelectrical Impedance (BI) method. Muscles, blood vessels and bones are body tissues with a high water content that conducts electricity easily. Body fat is tissue that has little electric conductivity. The BF206-BT sends an extremely weak electrical current of 50 kHz and less than 500 µA through your body to determine the amount of fat tissue. This weak electrical current is not felt while operating the BF206-BT.

In order for the scale to determine body composition, it uses the electrical impedance, along with your height, weight, age and gender information to generate results based on OMRON's data of body composition.

Recommended measurement times

Understanding the normal changes in your body fat percentage can help you in preventing or reducing obesity. Being aware of the times when the body fat percentages shift within your own daily schedule will assist you in obtaining an accurate trending of your body fat. It is recommended to use this unit in the same environment and daily circumstances. (See chart)

Avoid Taking Measurements Under the Following Conditions:

- Immediately after vigorous exercise, after a bath or sauna.
- After drinking alcohol or a large amount of water, after a meal (about 2 hours).

If a measurement is taken under these physical conditions, the calculated body composition may differ significantly from the actual one because the water content in the body is changing.

What is BMI (Body Mass Index)?

BMI uses the following simple formula to indicate the ratio between weight and height of a person.

$$\text{BMI} = \text{weight (kg)} / \text{height (m)} / \text{height (m)}$$

The OMRON BF206-BT uses the height information stored in your personal profile number or when entering information in the Guest Mode to calculate your BMI classification.

If the fat level revealed by BMI is higher than the international standard, there is an increased likelihood of common diseases. However, not all types of fat can be revealed by BMI.

Information on Body Composition

What is Body Fat Percentage?

Body fat percentage refers to the amount of body fat mass in regards to the total body weight expressed as a percentage.

$$\text{Body fat percentage (\%)} = \{\text{Body fat mass (kg)} / \text{Body weight (kg)}\} \times 100$$

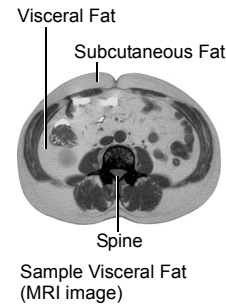
The device uses BI method to estimate your body fat percentage.

Depending on where fat is distributed in the body, it is classified as visceral fat or subcutaneous fat.

What is Visceral Fat Level?

Visceral fat = fat surrounding internal organs


Too much visceral fat is thought to be closely linked to increased levels of fat in the bloodstream, which can lead to common diseases such as hyperlipidemia and diabetes, which impairs the ability of insulin to transfer energy from the bloodstream and using it in cells. In order to prevent or improve conditions of common diseases, it is important to try and reduce visceral fat levels to an acceptable level. People with high visceral fat levels tend to have large stomachs. However, this is not always the case and high visceral fat levels can lead to metabolically obese. Metabolically obese (visceral obesity with normal weight) represents fat levels that are higher than average, even if a person's weight is at or below the standard for their height.

What is Subcutaneous Fat?

Subcutaneous fat = fat below the skin

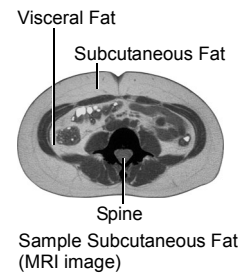Subcutaneous fat not only accumulates around the stomach but also around the upper arms, hips and thighs, and can cause a distortion of the body's proportions. Although not directly linked to increased risk of disease, it is thought to increase pressure on the heart and other complications. Subcutaneous fat is not displayed in this unit, but is included in the body fat percentage.

What is Skeletal Muscle?

Muscle is divided into two types, muscle in internal organs, such as the heart, and skeletal muscle attached to bones that is used to move the body. Skeletal muscle can be increased through exercise and other activity. Increasing the ratio of skeletal muscle means that body can burn energy more easily, which means that it is less likely to turn to fat, and makes it easier to lead an energetic lifestyle.

What is Resting Metabolism?

Regardless of your activity level, a minimum level of caloric intake is required to sustain the body's everyday functions. Known as the resting metabolism, this indicates how many calories you need to ingest in order to provide enough energy for your body to function.

What is Body Age?

Body age is based on your resting metabolism. Body age is calculated by using your weight, body fat percentage and skeletal muscle percentage to produce a guide to judge whether your body age is above or below the average for your actual age. To know your body age is useful to improve your health condition.

The reason calculated results may differ from actual body fat percentage

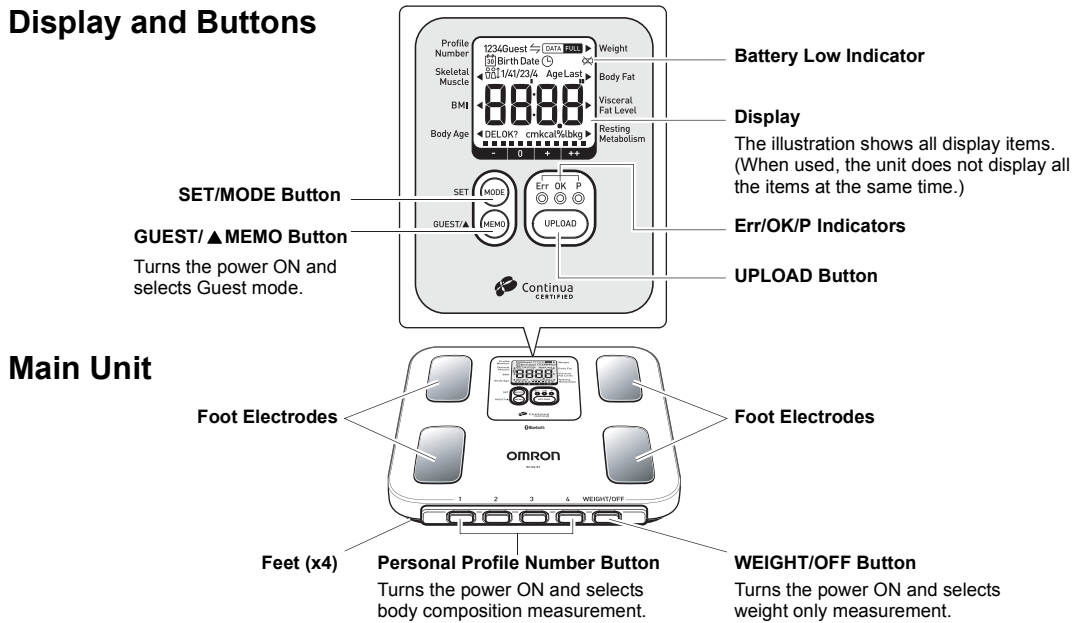
The body fat percentage measured by this unit may significantly differ from the actual body fat percentage in the following situations:

Elderly people (over 81 years old) / People with a fever / Body builders or highly trained athletes / Patients undergoing dialysis / Patients with osteoporosis who have very low bone density / Pregnant women / People with swelling

These differences may be related to changing ratios of body fluid and/or body composition.

1. Know Your Unit

Display and Buttons

About the Alarm

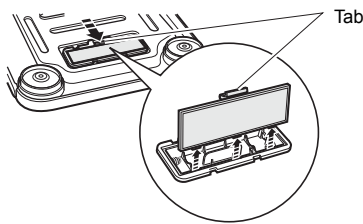
The alarm will beep under the following conditions:

- When the power is turned on: Blip!
- When 0.0 kg is displayed and the unit is ready to take a measurement: Blip! Blip!
- When a measurement is completed: Blip!
- When an error message "Err" appears: Blip! Blip! Blip! Blip! Blip! (A rapid series of short beeps.)
- When data has been sent successfully: Blip! Blip!
- When data failed to be sent: Beep! Beep! Beep! (A series of three long beeps.)

2. Inserting and Replacing the Batteries

1. Open the battery cover on the back of the unit.

- 1) Press the tab on the battery cover in the direction of the arrow to release the cover as illustrated.
- 2) Pull the tab of the cover upwards.


2. Install the batteries in correct polarity as marked inside the battery compartment.

3. Close the battery cover.

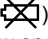Battery Life and Replacement

When the battery low indicator () appears, replace all four batteries with new ones.

Items stored in memory are retained even if the batteries are removed.

- Replace the batteries after turning off the power.
- Disposal of used batteries should be carried out in accordance with the national regulations for the disposal of batteries.
- Four AA alkaline batteries will last approximately six months (when measurements and data transfers are made four times a day).
- Because the supplied batteries are for trial use only, they may have a shorter life.

3. Setting the Date and Time

When the power is turned on for the first time, it is necessary to set the date and time.

- 1) Press the WEIGHT/OFF button to turn on the power.

When the power is turned on for the first time, the year digits blink.

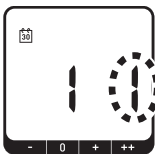
2. [TO SET THE YEAR]

- 1) Press the GUEST/▲MEMO button to adjust the year and press the SET/MODE button to confirm.
 - Setting range: 2010 to 2030
 - Press continuously to rapidly advance in increments of 10.

3. [TO SET THE MONTH AND DAY]

- 1) Press the GUEST/▲MEMO button to adjust the month and press the SET/MODE button to confirm.

- 2) Press the GUEST/▲MEMO button to adjust the day and press the SET/MODE button to confirm.
 - Press continuously to rapidly advance in increments of 10.

4. [TO SET THE HOUR AND MINUTE]

- 1) Press the GUEST/▲MEMO button to adjust the hour and press the SET/MODE button to confirm.
 - This is the 24-hour clock.

- 2) Press the GUEST/▲MEMO button to adjust the minute and press the SET/MODE button to confirm.
 - Press continuously to rapidly advance in increments of 10.

The current settings for the year, month and day, hour and minute appear in this sequence on the display. The power is automatically turned off.

This completes the setting.

Notes:

- If you need to correct while you are making settings: Press the WEIGHT/OFF button to turn the power off and on again. The year digits blink, proceed from step 2 of the procedure.
- If the power goes off while you are making settings: The power is automatically turned off if no information is entered for 3 minutes when setting the date and time. Press the WEIGHT/OFF button to turn the power off and on again.
- If you have to correct the date and time you have set: Press the WEIGHT/OFF button, and press the SET/MODE button for 2 seconds or longer after "0.0 kg" appears on the display. The year digits blink, proceed from step 2 of the procedure.

Power Switch

The power will automatically switch off in the following conditions:

1. If the monitor is not used within one minute of "0.0 kg" appearing on the display.
2. If no information is entered for 3 minutes when setting the date and time.
3. If no information is entered for 3 minutes when entering personal data.
4. If the monitor is not used for 3 minutes after the measurement results or last (previous) measurement results are displayed.
5. 30 seconds after the result is displayed when measuring weight only.
6. 10 seconds after "Err" appears on the display.

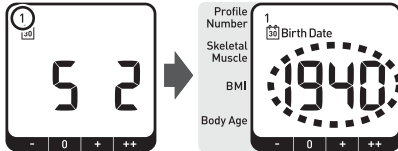
Note: "OFF" appears on the display when the power is turned off.

4. Setting and Storing Personal Data

For the correct measurement of body composition, it is necessary to set your personal data (birth date, gender, height). The monitor saves up to four personal profiles.

- 1) Press the Personal Profile Number button to select your personal profile number.

The current date appears on the display, then the display changes to the birth date (1940).


Note: When the date of birth, rather than current date, is displayed, the settings for that personal profile number are finished. Please use a different number or delete your personal profile information. Refer to Section 4, Deleting the Personal Data to delete the personal data.

2. [TO SET THE BIRTH DATE]


Setting range: 1900 to 2030

Press continuously to rapidly advance in increments of 10.

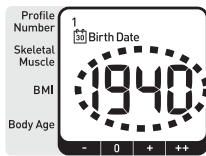
- 1) Press the GUEST/▲MEMO button to adjust the birth year and press the SET/MODE button to confirm.

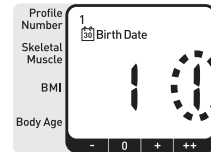- 2) Press the GUEST/▲MEMO button to adjust the birth month and press the SET/MODE button to confirm.

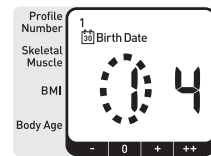
- 3) Press the GUEST/▲MEMO button to adjust the birth day and press the SET/MODE button to confirm.

3. [TO SET THE GENDER AND HEIGHT]

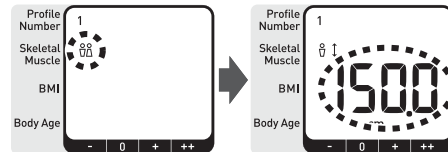Set the gender ♂ (MALE) or ♀ (FEMALE) and height in the same way.

After all the settings are displayed for your confirmation, "0.0 kg" appears on the display.

This completes the setting.

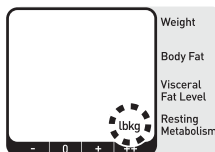
You can now take measurements of your body composition.

EN

Selecting Measurement Unit

You can change the measurement units used for the height and weight settings.

1. Press the WEIGHT/OFF button to turn on the power.
The current date appears on the display, then the display changes to "0.0 kg".
2. Keep the GUEST/▲MEMO button pressed for 2 seconds until "lb" and "kg" blink on the display.

3. Press the GUEST/▲MEMO button to select "kg" or "lb".
This will also automatically change "cm" and "inch".
4. Press the SET/MODE button to confirm.

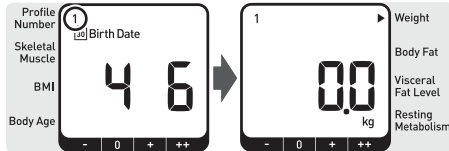
All height and weight measurements are displayed using the new measurement units. The change remains in effect until you change it again.

4. Setting and Storing Personal Data

Changing the Personal Data

1. Press the Personal Profile Number button to select your personal profile number.

The birth date appears on the display, then the display changes to "0.0 kg".
Wait until "0.0 kg" appears on the display.

2. Change the personal data.

- 1) While "0.0 kg" is displayed, press the SET/MODE button.

The birth year blinks on the display.

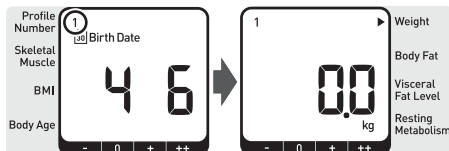
- 2) Refer to step 2 and 3 in Section 4, Setting and Storing Personal Data to change the personal data.

3. Press the WEIGHT/OFF button to turn off the power or take a measurement. (Refer to section 5 "Taking a Measurement".)

Deleting the Personal Data

1. Press the Personal Profile Number button to select the personal profile number you want to delete.

The birth date appears on the display, then the display changes to "0.0 kg".
Wait until "0.0 kg" appears on the display.


2. Delete the personal data.

- 1) While "0.0 kg" is displayed, press the SET/MODE button.


The birth year blinks on the display.

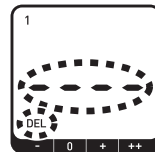
- 2) Press and hold the GUEST/▲MEMO button until "- - - -" and "DEL" blink on the display.

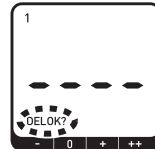- 3) Press the SET/MODE button.
"DEL OK?" blinks on the display.

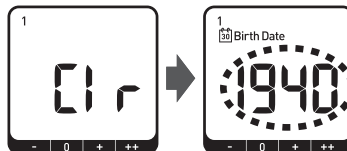- 4) Press the SET/MODE button.
"Clr" appears on the display and the birth year setting appears on the display.

Note: To cancel deleting, press the GUEST/▲MEMO button.

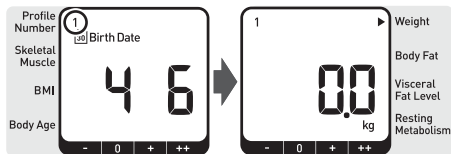
3. Press the WEIGHT/OFF button to turn off the power.

5. Taking a Measurement

Measurement should be taken on a level and hard surface.


1. Press the Personal Profile Number button to turn on the power.

The birth date appears on the display, then the display changes to "0.0 kg".

Note: If you step onto the unit before "0.0 kg" appears on the display, an error message "Err" will appear.

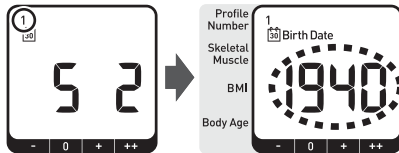If the following display is indicated:


Your personal data is not stored in the personal profile number you have selected. Please see Section 4 to store personal data.

If you do not have personal data stored on the unit (GUEST mode):


- 1) Press the GUEST/▲MEMO button to turn on the power. The current date appears on the display, then "GUEST" appears and the birth year (1940) blinks on the display.
- 2) Set the birth date, gender and height. Read steps 2 and 3 in Section 4, Setting and Storing Personal Data.

2. Start measurement.

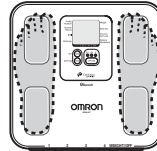- 1) Step on the unit bare-footed and place your feet on the foot electrodes with your weight evenly distributed.

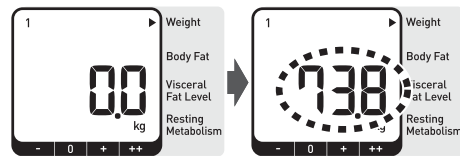
The display will show your weight and then the weight result will blink twice.

The unit will then start to measure your body composition.

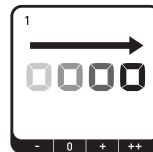- 2) During body composition measurement, "0" moves to show the measurement is progressing.

- 3) After the measurement is complete, your weight is displayed again. At this point you may step off the unit.

EN

Postures to Avoid During Measurement

Incorrect posture may result in inaccurate measurement of body composition.

Standing on edge of the unit

Knees bent

Crouching

5. Taking a Measurement

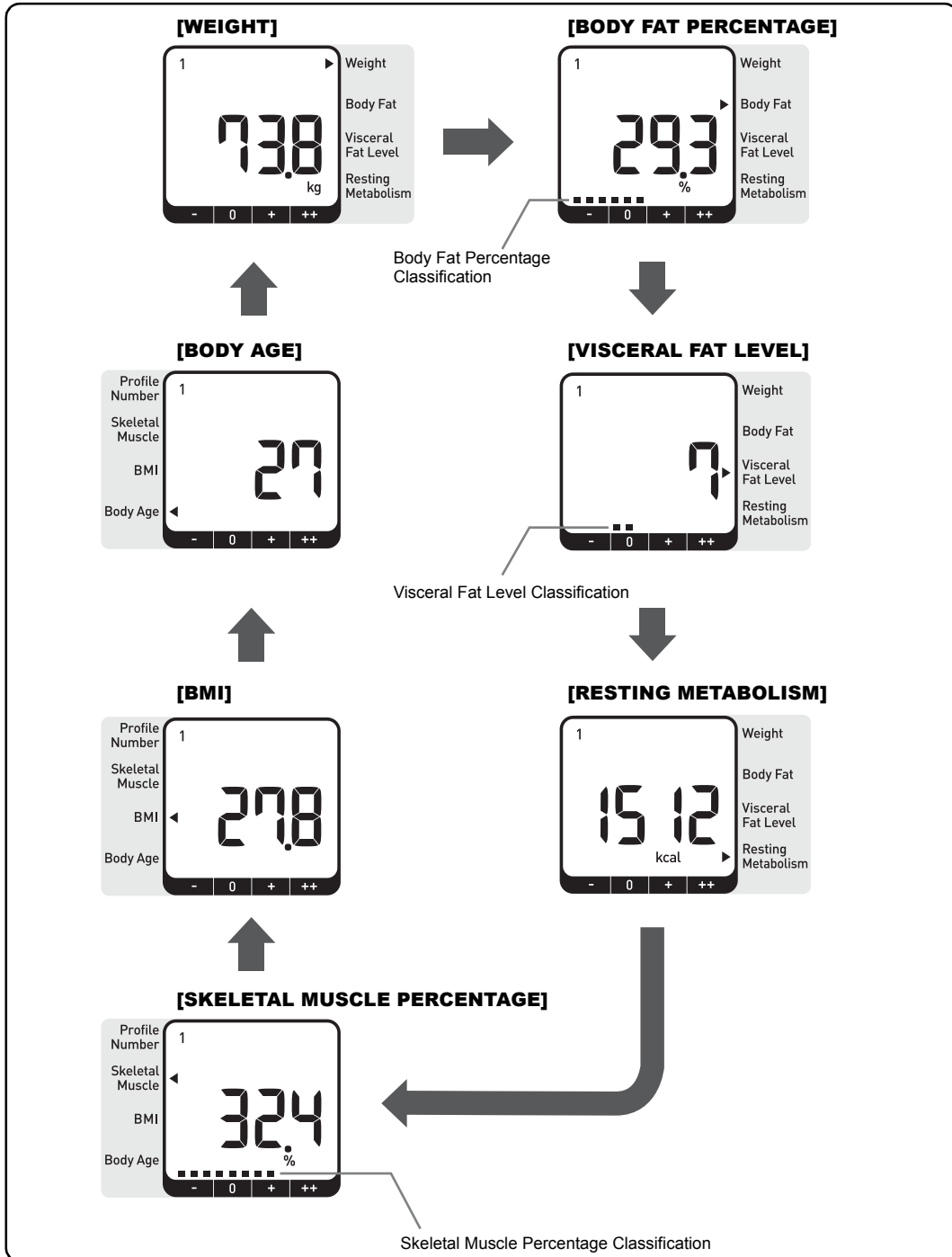
3. Check the measurement results.

The results display rotates automatically.

To stop the automatic rotation of results, or to view other measurement values, press the SET/MODE button.

The display changes with each press of the SET/MODE button, as shown below.

Note: Use the bar indicator as a guide to measurement.

Interpreting the Body Fat Percentage Result

Gender	Age	- (Low)	0 (Normal)	+ (High)	++ (Very High)
Female	20-39	< 21.0%	21.0 - 32.9%	33.0 - 38.9%	≥ 39.0%
	40-59	< 23.0%	23.0 - 33.9%	34.0 - 39.9%	≥ 40.0%
	60-80	< 24.0%	24.0 - 35.9%	36.0 - 41.9%	≥ 42.0%
Male	20-39	< 8.0%	8.0 - 19.9%	20.0 - 24.9%	≥ 25.0%
	40-59	< 11.0%	11.0 - 21.9%	22.0 - 27.9%	≥ 28.0%
	60-80	< 13.0%	13.0 - 24.9%	25.0 - 29.9%	≥ 30.0%

Based on NIH/WHO guidelines for BMI
 Based on Gallagher et al., American Journal of Clinical Nutrition, Vol.72, Sept. 2000

Interpreting the Visceral Fat Level Result

Visceral Fat Level	Level Classification
1 - 9	0 (Normal)
10 - 14	+ (High)
15 - 30	++ (Very High)

According to OMRON HEALTHCARE figures

Interpreting the BMI Result

BMI	BMI (Designation by the WHO)
BMI < 18.5	Underweight
18.5 ≤ BMI < 25	Normal
25 ≤ BMI < 30	Overweight
30 ≤ BMI	Obese

The above-mentioned indices refer to the values for obesity judgment proposed by WHO, the World Health Organization.

Interpreting the Skeletal Muscle Percentage Result (for adults)

Gender	Age	- (Low)	0 (Normal)	+ (High)	++ (Very High)
Female	18-39	< 24.3%	24.3 - 30.3%	30.4 - 35.3%	≥ 35.4%
	40-59	< 24.1%	24.1 - 30.1%	30.2 - 35.1%	≥ 35.2%
	60-80	< 23.9%	23.9 - 29.9%	30.0 - 34.9%	≥ 35.0%
Male	18-39	< 33.3%	33.3 - 39.3%	39.4 - 44.0%	≥ 44.1%
	40-59	< 33.1%	33.1 - 39.1%	39.2 - 43.8%	≥ 43.9%
	60-80	< 32.9%	32.9 - 38.9%	39.0 - 43.6%	≥ 43.7%

According to OMRON HEALTHCARE figures

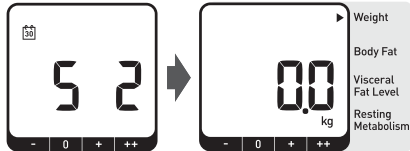
EN

4. Press the WEIGHT/OFF button to turn off the power.

6. Measuring Weight Only

1. Press the WEIGHT/OFF button to turn on the power.

The current date appears on the display, then the display changes to "0.0 kg".

Note: If you step onto the unit before "0.0 kg" appears on the display, an error message "Err" will appear.

2. When the "0.0 kg" is displayed, step onto the unit.

3. Check the measurement result.

Your weight is displayed and blinks twice to indicate that measurement is completed.

4. When the measurement is completed, step down from the unit and turn off the power.

7. Viewing the Last Measurement Results

Up to 30 measurement results are automatically stored for each personal profile number and the last (previous) measurement results can be displayed for this unit.

Note: When using the Guest mode, measurement results are not stored.

1. Press the Personal Profile Number button to turn on the power.

2. Take a measurement.

If you want to compare the results with the last (previous) values, measure your body composition.

3. Press GUEST/▲MEMO button.

The last weight value is displayed.

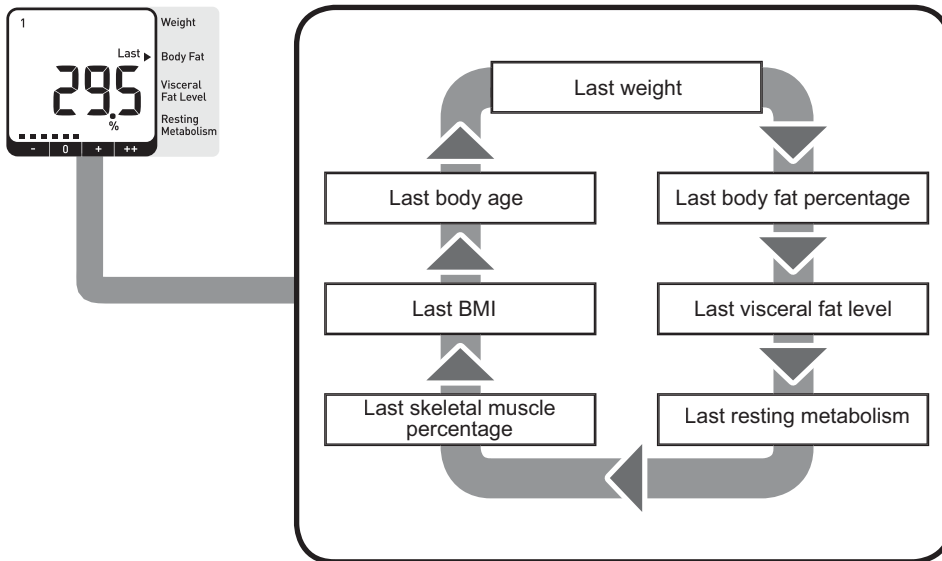
If you have measured your body composition in step 2, each press of the GUEST/▲MEMO button alternates between the current measurement result and the last (previous) result.

Notes:

- If there are no results for the last measurement, “- - - -” is displayed.
- If you have not measured your body composition in step 2, current measurement results cannot be displayed even if you press the GUEST/▲MEMO button.

4. Press the SET/MODE button to cycle through the other measurement values.

Note: Refer to Section 8 if you want to transfer the data.

5. Press the WEIGHT/OFF button to turn off the power.

8. Transferring the Data

You can transfer measurement data to a *Bluetooth*® wireless technology device. You can transfer up to 30 sets of stored measurement results for each personal profile number. The content and display for the measurement results may vary depending on the application used to view them. For details, refer to the manual provided with the application you are using.

Registering the Unit with the Destination Device (Pairing)

What is Pairing:

This refers to the procedure for registering the unit with the device so that it can be used to transfer data. Pairing is required when a device is connected to the unit for the first time, or if the same device is used to transfer data for a different personal profile number. Follow the procedure below to pair the unit with the destination device.

Refer to “Transferring the Data” for information about transferring the data with a device that has already been paired with the unit.

1. Display the measurement results.

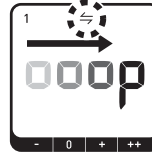
- Pairing (Registration) after taking a measurement:
Follow the steps in Section 5, to display the measurement results.
- Pairing (Registration) without taking a measurement:
Follow the steps in Section 7, to display the results for the last measurement.
- Pairing (Registration) when using GUEST mode:
You can also register a device after taking a measurement using GUEST mode. Follow the steps in Section 5, to display the measurement results.

2. Enable pairing (registration) on the device.

Set the application on the device ready for pairing.

For details, refer to the manual provided with the application you are using.


3. Press the UPLOAD button for 2seconds or longer.

Use the application to pair the unit with the device. For details, refer to the manual provided with the application you are using.

Note: If you press the UPLOAD button while pairing, pairing will be stopped and the display will revert to the previous screen.

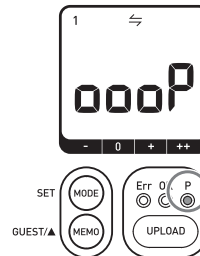If the pairing completes successfully, the “P” indicator lights and the alarm will beep (Blip! Blip!).

Notes:

- If the pairing process does not complete within approximately one minute, it will end without pairing the unit with the device.
- If the pairing process does not complete successfully, then the “Err” indicator lights and the alarm will beep three times (Beep! Beep! Beep!). Check the application on the connected device, then repeat the procedure from step 2 to try again.

Transferring the Data

1. Display the measurement results.

- Transferring data after taking a measurement:
Follow the steps in Section 5, to display the measurement results.
- Transferring data without taking a measurement:
Follow the steps in Section 7, to display the results for the last measurement.
- Transferring data when using GUEST mode:
You can also register a device after taking a measurement using GUEST mode.
Follow the steps in Section 5, to display the measurement results.

Note: When using GUEST mode, you cannot send results stored on the unit, you can only send the measurement results for the current measurement.

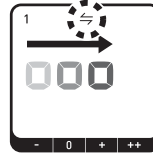
2. Enable data transfers.

Set the application on the destination device to transfer mode.
For details, refer to the manual provided with the application you are using.



3. Press the UPLOAD button.

The display changes to the data transmission screen, and the unit starts transferring the data.

Note: If you press the UPLOAD button while the data is being transferred, the transfer will be stopped and the display will revert to the previous screen.

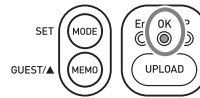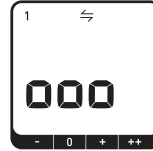If the data is sent successfully, the "OK" indicator lights and the alarm will beep (Blip! Blip!).


Note: If the data transmission fails, then the "Err" indicator lights and the alarm will beep three times (Beep! Beep! Beep!).
Check the application on the connected device, then repeat the procedure from step 2 to try again.

The screen will return to the screen displayed before data transmission started after 10 seconds.

4. Press the WEIGHT/OFF button to turn off the power.

About Data FULL Indicator

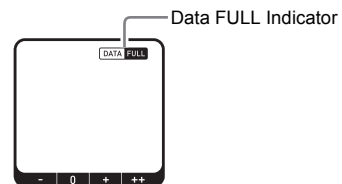Up to 30 sets of measurement results can be stored for each personal profile number.
If the number exceeds 30, the oldest results are deleted.
If the Data FULL indicator blinks or remains lit, transfer data to a device as soon as possible to avoid losing measurement results.

DATA FULL blinks: 24-29 sets of measurement results have been stored.

DATA FULL lit: 30 sets of measurement results have been stored.

Note: The Data FULL indicator is not displayed when only 0 to 23 measurements results are stored.

EN

9. About the Wireless Communications Function

CE0197

Hereby, OMRON HEALTHCARE Co.,Ltd, declares that this Body Composition Monitor is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC. The declaration of conformity may be consulted at www.omron-healthcare.com.

This Product operates in the unlicensed ISM band at 2.4GHz. In case this Product is used around the other wireless devices including microwave oven and wireless LAN, which operate same frequency band of this Product, there is a possibility that interference occurs between this Product and such other devices. If such interference occurs, please stop the operation of other devices or relocate this Product before using this Product or do not use this Product around the other wireless devices.

The *Bluetooth*® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by OMRON HEALTHCARE Co.,Ltd. is under license. Other trademarks and trade names are those of their respective owners.

Look for the Continua Certified logo. It's your assurance that a device will reliably communicate and exchange information with similarly certified products, to help you better manage your health and wellness.

10. How to Take Care of and Store the Unit

This *Bluetooth*® Body Composition Monitor does not require any maintenance.

How to Clean the Unit

- Always keep the unit clean before use.
- Wipe the unit with a soft dry cloth.
 - If necessary, use a cloth moistened with water or detergent and squeeze it well before wiping the unit, then wipe dry with a dry cloth.
- You can use cleaning alcohol to clean the electrodes, but do not use it on other parts of the unit.
- Do not use benzene or thinner, to clean the unit.

Care and Storage

- Do not store the unit in the following conditions:
 - Humidity, where moisture or water may get into the unit
 - High temperatures, direct sunlight or dusty places
 - Places with the risk of sudden shocks or vibrations
 - In places where chemicals are stored or where corrosive gas is present.
- Do not carry out repairs of any kind yourself. If a defect occurs, consult the OMRON distributor or Customer Services as mentioned on the packaging.

11. Error Displays

Error Display	Cause	Correction
Err 1	Your soles are not in firm contact with the electrodes.	Press your soles firmly to the electrodes, then measure. (Refer to Section 5.)
Err 2	The posture for measurement is wrong or the soles are not in firm contact with the electrodes.	Measure without moving the soles. (Refer to Section 5.)
Err 3	The soles are too dry.	Moisten the soles with a wet towel, then repeat measurement.
Err 5	Device error.	Insert the batteries again and repeat measurement. If this error still occurs, consult your OMRON service representative.
Err	You stepped on or moved the unit before the display indicated "0.0 kg".	Step on the unit after the display indicates "0.0 kg".
	You moved your body while measuring body weight.	Do not move while measuring body weight.
	Your weight is more than 150 kg (330.0 lb).	Body weight of more than 150 kg (330.0 lb) is out of the measuring range of this unit. You cannot use this unit.

12. Troubleshooting


Problem	Cause	Correction
The displayed value of body composition is abnormally high or low.	Refer to "Information on Body Composition".	
"- - -" is displayed for some items of the measurement results.	Values of body composition are out of measurable range.	<ul style="list-style-type: none"> • Please ensure that birth date, gender and height settings stored as personal data are correct. (Refer to Setting Items in Section 13.) • The unit cannot measure body composition outside its measurement range, even if the settings are correct.
The power turns off after the body weight is confirmed and before your body fat percentage, visceral fat level, skeletal muscle percentage, BMI and resting metabolism is measured.	You did not select the correct personal profile number or the GUEST mode. (The personal profile number or "GUEST" is not displayed on the display unit.)	Select the correct personal profile number or the GUEST mode. (Refer to Section 5.)
Nothing is displayed when the power is turned on.	Batteries are not inserted.	Insert the batteries.
	The polarities of batteries are not aligned correctly.	Insert the batteries in correct alignment.
	The batteries are worn out.	Replace all four batteries with new ones. (Refer to Section 2.)
Data cannot be sent. ("Err" indicator lights.)	The destination device for the data is too far away.	After checking that there are no sources of interference nearby, move the device to a distance within 5 m of the unit.
	Pairing (registering) has not been completed.	Perform pairing (registration). (Refer to Section 8.)
	The application on the connected device is not ready.	Check the application then try sending the data again. If the "Err" indicator still lights after checking the application, contact the service (support) center of the application provider.

EN

13. Technical Data

Product	<i>Bluetooth</i> [®] Body Composition Monitor	
Model	BF206-BT (HBF-206IT-E)	
Display	Body Weight:	0 to 150 kg with an increment of 0.1 kg (0.0 to 330.0 lb with an increment of 0.2 lb)
	Body Fat percentage:	5.0 to 60.0% with an increment of 0.1%
	Skeletal Muscle percentage:	5.0 to 50.0% with an increment of 0.1%
	BMI:	7.0 to 90.0 with an increment of 0.1
	Resting Metabolism:	385 to 3999 kcal with an increment of 1 kcal
	Body Age:	18 to 80 years old with an increment of 1 year
	Visceral Fat Level:	30 levels with an increment of 1 level
	Body fat percentage and Skeletal muscle percentage classification:	- (Low) / 0 (Normal) / + (High) / ++ (Very High) 4 levels with 3 sublevels each
	Visceral fat level classification:	0 (Normal) / + (High) / ++ (Very High) 3 levels with 3 sublevels each
Memory	Display memory:	Last (previous) measurement results
	Unit stored memory:	Up to 30 sets
Transmission Protocol	<i>Bluetooth</i> [®] wireless technology Version 2.1 + EDR Class 2	
Setting Items	Personal data information:	The following information for up to 4 people can be stored.
	Height:	100.0 to 199.5 cm (3' 4" to 6' 6 3/4")
	Birth Date:	1st January, 1900 to 31st December, 2030
	Gender:	Male/Female
	* Measurement Units: kg (cm) / lb (feet+inch)	
	* The age range for the Body Fat percentage is 10 to 80 years old.	
	* The age range for the Skeletal Muscle percentage, Skeletal Muscle percentage classification, Resting Metabolism rate, Visceral Fat level, Visceral Fat level classification and Body age is 18 to 80 years old.	
	* The age range for the Body Fat percentage classification is 20 to 80 years old.	
Weight Accuracy	0.0 kg to 40.0 kg: ± 0.4 kg (0.0 lb to 88.2 lb: ± 0.88 lb)	
	40.0 kg to 150.0 kg: ± 1% (88.2 lb to 330.0 lb: ± 1%)	
Accuracy (S.E.E.)	Body Fat percentage:	3.5%
	Skeletal Muscle percentage:	3.5%
	Visceral Fat Level:	3 levels
Power Supply	4 AA alkaline batteries (LR6)	
Battery Life	Approximately six months (When AA alkaline batteries are used with four measurements and data transfers a day at a room temperature of 23°C.)	
Operating Temperature / Humidity	+10°C to +40°C, 30% to 85% RH	
Storage Temperature/ Humidity/Air Pressure	-20°C to +60°C, 10% to 95% RH, 700 hPa - 1060 hPa	
Weight	Approximately 1.8 kg (3.97 lb) (including batteries)	
External Dimensions	Approx. 300 (W) × 38 (H) × 290 mm (D) (Approx. 11 3/4" (W) × 1 1/2" (H) × 11 3/8" (D))	
Package Contents	<i>Bluetooth</i> [®] Body Composition Monitor, 4 AA alkaline batteries (LR6), instruction manual, Guarantee Card	

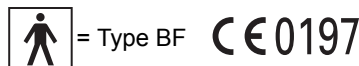Note: Subject to technical modification without prior notice.

This device fulfills the provisions of the EC directive 93/42/EEC (Medical Device Directive).

**Correct Disposal of this Product
(Waste Electrical & Electronic Equipment)**

This marking shown on the product or its literature, indicates that it should not be disposed with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources.

Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this item for environmentally safe recycling.

Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial wastes for disposal.

This product does not contain any hazardous substances.

Disposal of used batteries should be carried out in accordance with the national regulations for the disposal of batteries.

Important information regarding Electro Magnetic Compatibility (EMC)

With the increased number of electronic devices such as PCs and mobile (cellular) telephones, medical devices in use may be susceptible to electromagnetic interference from other devices.

Electromagnetic interference may result in incorrect operation of the medical device and create a potentially unsafe situation.

Medical devices should also not interfere with other devices.

In order to regulate the requirements for EMC (Electro Magnetic Compatibility) with the aim of preventing unsafe product situations, the EN60601-1-2 standard has been implemented. This standard defines the levels of immunity to electromagnetic interferences as well as maximum levels of electromagnetic emissions for medical devices.

This medical device manufactured by OMRON HEALTHCARE conforms to this EN60601-1-2:2007 standard for both immunity and emissions.

Nevertheless, special precautions need to be observed:

Do not use mobile (cellular) telephones and other devices, which generate strong electrical or electromagnetic fields, near the medical device. This may result in incorrect operation of the unit and create a potentially unsafe situation. Recommendation is to keep a minimum distance of 7 m.

Verify correct operation of the device in case the distance is shorter.

Further documentation in accordance with EN60601-1-2:2007 is available at OMRON HEALTHCARE EUROPE at the address mentioned in this instruction manual.

Documentation is also available at www.omron-healthcare.com.

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