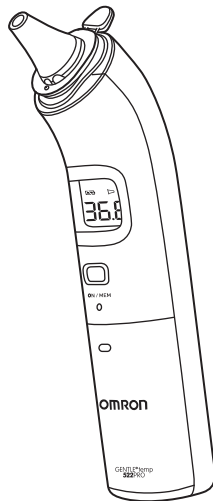
## Digital Ear Thermometer Gentle Temp 522 PRO Instruction Manual

All for Healthcare

English

Français

Deutsch

Italiano

Español

Nederlands

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Türkçe

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# Contents

Thank you for purchasing the OMRON Gentle Temp 522 PRO Digital Ear Thermometer.

## Intended Use:

The OMRON Gentle Temp 522 PRO offers comfortable, safe, accurate, and quick temperature measurement from the eardrum.

It is mainly designed for professional use.

<b>Important Safety Information</b> .....	<b>1</b>	<b>4. Troubleshooting and Maintenance</b> .....	<b>13</b>
<b>1. Overview</b> .....	<b>3</b>	4.1 The Icons and Error Messages .....	13
<b>2. Preparation</b> .....	<b>4</b>	4.2 Maintenance .....	14
2.1 Removing the Insulating Tape .....	4	4.3 Replacing the Battery .....	15
2.2 Switching between °C and °F .....	5	<b>5. Technical Data</b> .....	<b>16</b>
2.3 Setting the Buzzer .....	6	<b>6. Some Useful Information</b> .....	<b>19</b>
2.4 Attaching a Probe Cover .....	7	6.1 Temperature Measurements Taken within the Ear .....	19
2.5 Detaching a Probe Cover .....	8	6.2 Knowing a Patient's Normal Temperature .....	19
<b>3. Using the Unit</b> .....	<b>9</b>	6.3 Ear Temperature Compared to Other Types of Body Temperature .....	20
3.1 Taking a Reading .....	9	6.4 Questions and Answers .....	21
3.2 Using the Memory Function .....	12	<b>7. Optional Accessories</b> .....	<b>22</b>

**Please read this instruction manual thoroughly before using this unit.**

**Please keep for future reference.**

# Important Safety Information

To assure correct use of the product, basic safety measures should always be followed, including the precautions listed below.

**⚠ Warning: The following situations are potentially hazardous for your patients and, if not avoided, could result in death or serious injury.**

- Keep still during measurement.
- Do not forcibly insert the probe in the ear.
- If your patients feel discomfort such as a pain during the measurement, stop using the unit immediately. It may injure the external auditory canal.
- Do not use this unit if suffering from ear disease such as otitis externa or otitis media. It may worsen the condition.
- Do not use this unit when the external auditory canal is wet, such as after the patient has been swimming or taken a bath. It may injure the external auditory canal.
- Do not use this unit without attaching a probe cover.
- Do not use this unit near an electronic device or magnetic card, such as a computer, credit card, etc. Doing so may damage the device or card.
- Ensure that the ear canal is clean and free of earwax. Correct measurement results may not be obtained if dirty probe covers are used.
- Do not use one probe cover twice. This can lead to cross infections such as external otitis. Always use a clean, undamaged, and new probe cover.
- Proper installation of the probe cover ensures accurate measurements.
- When the probe lens becomes dirty, wipe it lightly with a soft dry cloth or a cotton swab. Do not wipe the probe lens with tissue paper or a paper towel. For details, refer to section 4.2.
- Do not use more than one probe cover at a time.
- If there is any temperature difference between the places where the unit is stored and where it will be used to measure, leave the unit in the room where it will be used for more than 30 minutes to allow it to reach room temperature first, then measure.
- If the patient's ear is cold, wait until the ear is warmed up before taking a temperature measurement. The measured result may indicate low when the patient uses an ice bag or an ice pack, or immediately after coming in from the outside in winter.

## Important Safety Information

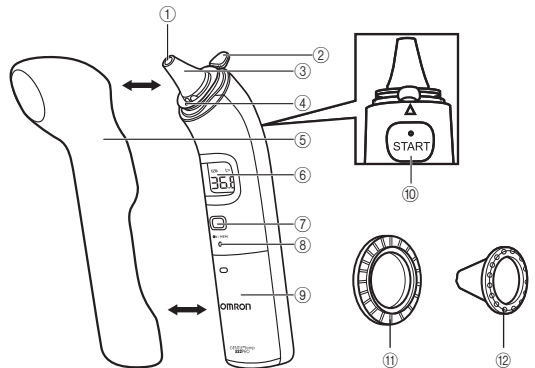
- Do not touch the probe lens with a finger or breathe on it. Doing so may soil or damage the probe lens.
- Do not attempt measurements when the unit is wet as inaccurate readings may result.
- Check the icon on the display before and after the measurement so that the measurement is taken in the appropriate mode.
- Keep the unit out of children's reach.
- Do not throw batteries into a fire. The battery may explode.
- Remove the battery when the unit will not be used for 3 months or more. Failure to do so may lead to fluid leakage, heat generation or bursting, resulting in damage to the unit.
- Do not use the unit in places where strong static electricity or electromagnetic fields are present. Doing so may lead to inaccurate readings and may contribute to instrument failure.

### **General Precautions**

- Do not use this unit other than for measuring the temperature in the human ear.
- Do not apply a strong shock to, drop, step on, or vibrate the main unit.
- Do not use a mobile phone near the unit.
- The main unit is not waterproof. Be careful when handling this unit so that no liquid (alcohol, water, or hot water) will get into the main unit. When the unit is wet with vapor, wait until it dries or wipe it lightly with a soft dry cloth.
- Do not disassemble, repair, or modify the unit.

# 1. Overview


Main unit:

- ① Probe lens
- ② Probe cover ejector
- ③ Probe
- ④ Probe cover detector
- ⑤ Hard case
- ⑥ Display

- ⑦ ON/MEM button
- ⑧ Battery cover pick hole
- ⑨ Battery compartment
- ⑩ START button
- ⑪ Connection ring
- ⑫ Probe cover

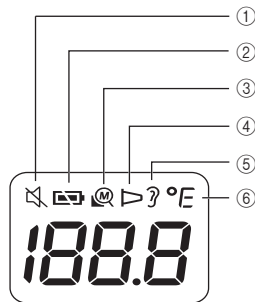Display:

- ① Buzzer
- ② Battery icon
- ③ Memory icon
- ④ Probe cover icon
- ⑤ Ear icon
- ⑥ Temperature mode


## 2. Preparation
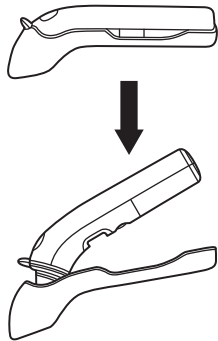
### 2.1 Removing the Insulating Tape


---

1. Gently remove the unit from its hard case.

**Note:** Do not forcibly remove the unit.

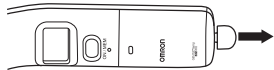
- 
2. When using the thermometer for the first time, pull the insulating tape out of the battery compartment.

## 2.2 Switching between °C and °F

This unit is set in °C as default.

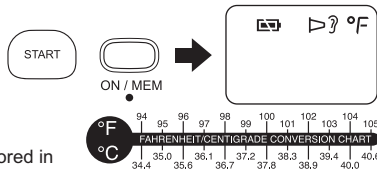1. While the power is off, press and hold the **START** button.
2. While holding it down, press and hold the **ON/MEM** button until °F appears on the display with two beeps.

**Notes:**

- To select °C mode again, start from step 1.
- When the unit is switched between °C and °F, the last reading stored in the memory is deleted.

### 2.3 Setting the Buzzer

The buzzer is activated by default, but can be deactivated.

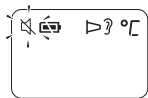
---

**1. Press the ON/MEM button to turn on the unit.**

---

**2. Press and hold the ON/MEM button for 3 seconds.**


The “” icon flashes on the display.

---

**3. Release the ON/MEM button.**

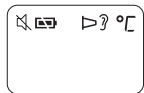The “” icon remains lit and the buzzer is deactivated.

**Notes:**

- If the ON/MEM button is pressed down for more than 5 seconds after the “” icon starts flashing, the unit turns off without deactivating the buzzer.
- To activate the buzzer again, start from step 1.

### 2.4 Attaching a Probe Cover

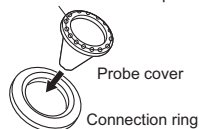
Always use a new and undamaged OMRON MC-EP3-E probe cover.

#### 1. Place a new probe cover on the connection ring.

**Notes:**

- The adhesive side of probe cover should be upward.
- A probe cover should be used once only, to prevent cross infection.



Place adhesive side upward

#### 2. Insert the probe into the probe cover on the connection ring until it clicks.

**Notes:**

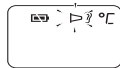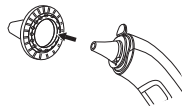- When the probe cover is not attached correctly, the probe cover icon “▷” will flash on the display and a measurement cannot be taken.
- Using the OMRON optional MC-EPL1-E probe cover loader allows for fast and easy attaching of a probe cover. Please refer to section 7.
- Please refer to section 7 for the probe cover and probe cover loader.

## 2.5 Detaching a Probe Cover

---

**Push the probe cover ejector to remove the probe cover.**

**Note:** When removing a probe cover, gently hold its edge with your other hand to prevent it springing off.

# 3. Using the Unit

## 3.1 Taking a Reading

### Notes:

- Make sure the probe cover is attached correctly.
- It is recommended to perform measurement three times with the same ear. If the three measurements are different, select the highest temperature.

EN

### 1. Press the ON/MEM button.

All indicators appear on the display.

Then the display shown on the far right appears with two beeps.

### 2. Insert the probe into the ear as far as it comfortably goes in the direction of the eardrum.

### Notes:

- Gently pull the ear back to straighten the ear canal and position the probe into the ear so it is snug, aiming towards the membrane of the eardrum to obtain an accurate reading.
- Holding the unit too long in the hand may cause a higher ambient temperature reading of the probe. This could make the body temperature measurement lower than usual.

### 3. Using the Unit

#### Measuring the temperature of an infant

Measuring on a lying baby.

Lightly support the child's body.

Measuring on a sitting baby.

Lightly support the child's body and slightly pull the ear towards the back.

The ear is too small to insert the probe.

While slightly pulling the ear back, cover the external auditory canal with the probe without forcibly trying to insert the probe.

#### 3. Press the **START** button.

The measurement is complete in 1 second with a long beep.

EN

#### 4. Remove the unit from the ear and check the measurement result.

The “?” icon flashes for 5 seconds.

**Notes:**

- If the measurement result is over 37.5°C (99.5°F), the buzzer beeps three times after a long beep.
- Another measurement can be taken after the “?” icon stops flashing with two beeps. Make sure the “?” icon remains lit.

#### 5. Press and hold the **ON/MEM** button until “OFF” appears on the display to turn off the unit.

The unit automatically stores the measurement in its memory.

The unit also turns off automatically if no operation is performed for 30 seconds.

## 3.2 Using the Memory Function

The unit automatically stores the previously measured value.

---

**1. Press the ON/MEM button to turn on the unit.**

---

**2. Press the ON/MEM button again.**

The memory number appears on the display.

---

**3. Release the ON/MEM button.**

The previously measured value appears on the display.

**Note:** If there is no measurement record in this unit, the lower right screen will be displayed.

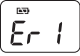
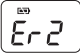
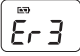
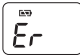
---

**4. Press and hold the ON/MEM button until “OFF” appears on the display to turn off the unit.**

The unit also turns off automatically if no operation is performed for 30 seconds.

# 4. Troubleshooting and Maintenance

## 4.1 The Icons and Error Messages

Error Display	Cause	Remedy
	Unit stabilization in process.	Wait until  stops flashing.
	Battery is low.	Replace the battery. (Refer to section 4.3)
	Probe cover is not attached correctly.	Attach the probe cover again until  stops flashing.
	Measurement before unit stabilization.	Wait until  stops flashing.
	The unit is showing a rapid ambient temperature change.	Allow the unit to rest in a room for at least 30 minutes at room temperature: 10°C to 40°C (50°F to 104°F).
	The ambient temperature is not within the range between 10°C to 40°C (50°F to 104°F).	Allow the unit to rest in a room for at least 30 minutes at room temperature: 10°C to 40°C (50°F to 104°F).
	Errors 5–9 indicate a malfunction of the unit.	Remove the battery, wait for 1 minute, reinsert the battery, and turn on the unit again. If the message reappears, please contact the OMRON retail outlet or distributor for having the unit checked.

EN

## 4. Troubleshooting and Maintenance

Error Display	Cause	Remedy
	Temperature taken is higher than 42.2°C (108.0°F).	Check the integrity of the probe cover and take a new temperature measurement.
	Temperature taken is lower than 34.0°C (93.2°F).	Make sure the probe cover is clean and take a new temperature measurement.
	Unit cannot be powered on to ready stage.	Change to a new battery. (Refer to section 4.3)

### 4.2 Maintenance

- Check the unit if it is dropped. If unsure, please contact the OMRON retail outlet or distributor for having the unit checked.
- The probe is the most delicate part of the unit. Use care when cleaning the probe lens to avoid damage. When the probe lens becomes dirty, wipe it lightly with a soft dry cloth or a cotton swab.
- If the unit is accidentally used without the probe cover, clean the probe as follows:
  - a. After measurement, use a cotton swab moistened with alcohol (70% concentration) to clean the probe and probe lens.
  - b. Allow at least 1 minute for the probe to fully dry.
- If the unit is dirty, use a cotton swab or cloth moistened with alcohol (70% concentration) to clean it.
- Do not store the unit in the following types of places. Doing so may damage the unit.
  - Wet locations.
  - Locations with high heat and humidity or those that are exposed to direct sunlight. Areas close to heating equipment, dusty locations, or environments where there are high salt concentrations in the air.
  - Locations where the unit will be subjected to leaning over, falling, shock, or vibration.
  - Locations where corrosive gases are present.
  - Pharmaceutical storage areas.

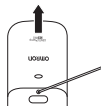
### 4.3 Replacing the Battery

Battery: CR2032 lithium button battery

**Note:** To protect the environment, discard the used batteries in accordance with the local regulations regarding waste disposal procedure. Disposal can be done at your retail store or at appropriate collection sites.

EN

- 1. Insert a pointed object into the battery cover pick hole. Slide and remove the battery cover with your thumb.**

- 2. Remove the battery with a pointed object.**

**Note:** Do not use a metal tool as it may damage the surrounding parts.

- 3. Insert the new battery under the metal hook on the left side and press the right side of the battery down until it clicks.**

**Note:** Replace the new battery with the plus (+) side facing out.

- 4. Replace the battery cover.**

# 5. Technical Data

<b>Product Description:</b>	OMRON Digital Ear Thermometer
<b>Model:</b>	Gentle Temp 522 PRO (MC-522-E)
<b>Sensor:</b>	Thermopile
<b>Temperature Display:</b>	4-digit, °F display in 0.1 degree increments 3-digit, °C display in 0.1 degree increments
<b>Accuracy:</b>	±0.2°C (±0.4°F) within 35.0°C to 42.0°C (95.0°F to 107.6°F), ±0.3°C (±0.5°F) for other range
<b>Measurement Range:</b>	34.0°C (93.2°F) to 42.2°C (108.0°F)
<b>Power Supply:</b>	3.0V DC, 1 CR2032 lithium button battery
<b>Power Consumption:</b>	0.015 W
<b>Battery Life:</b>	With a new battery, approx. 6,000 measurements or more
<b>Operating Temperature and Humidity:</b>	10°C (50°F) to 40°C (104°F), 15 to 85%RH
<b>Storage Temperature/Humidity/Air Pressure:</b>	-20°C (-4°F) to 60°C (140°F), 10 to 95%RH, 200 to 1060hPa
<b>Protection against electric shock:</b>	Internally powered ME equipment
<b>Applied Part:</b>	 = type BF
<b>Weight:</b>	Approx. 63 g (with battery installed)
<b>Outer Dimensions:</b>	36 (w) × 154 (h) × 48 (d) mm

## 5. Technical Data

**Package Contents:** Main unit, test battery (lithium button battery CR2032 installed), hard case, probe cover, connection ring, instruction manual.

### Notes:

- The specification may be changed without prior notice.
- This OMRON product is produced under the strict quality system of OMRON HEALTHCARE Co. Ltd., Japan.
- This device fulfills the provisions of the EC directive 93/42/EEC (Medical Device Directive) and the European Standards EN12470-5:2003 and EN ISO 80601-2-56:2012: Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement.

**CE0197**

OMRON HEALTHCARE EUROPE B.V. guarantees this product for 3 years after date of purchase. The guarantee does not cover battery, packaging and/or damages of any kind due to misuse (such as dropping or physical misuse) caused by the user. Claimed products will only be replaced when returned together with the original invoice/cash ticket. Do not disassemble or modify the unit. Changes or modifications not approved by OMRON HEALTHCARE will void the user warranty.

## 5. Technical Data

### OMRON Digital Ear Thermometer

#### **Important information regarding Electro Magnetic Compatibility (EMC)**

With the increased number of electronic devices such as PC's and mobile (cellular) telephones, medical devices in use may be susceptible to electromagnetic interference from other devices. Electromagnetic interference may result in incorrect operation of the medical device and create a potentially unsafe situation.

Medical devices should also not interfere with other devices.

In order to regulate the requirements for EMC (Electro Magnetic Compatibility) with the aim to prevent unsafe product situations, the EN60601-1-2 standard has been implemented. This standard defines the levels of immunity to electromagnetic interferences as well as maximum levels of electromagnetic emissions for medical devices.

This medical device manufactured by OMRON Healthcare conforms to this EN60601-1-2:2007 standard for both immunity and emissions.

Nevertheless, special precautions need to be observed:

- Do not use mobile (cellular) telephones and other devices, which generate strong electrical or electromagnetic fields, near the medical device. This may result in incorrect operation of the unit and create a potentially unsafe situation. Recommendation is to keep a minimum distance of 7 m. Verify correct operation of the device in case the distance is shorter.

Further documentation in accordance with EN60601-1-2:2007 is available at OMRON HEALTHCARE EUROPE at the address mentioned in this instruction manual.

Documentation is also available at [www.omron-healthcare.com](http://www.omron-healthcare.com).

#### **Correct Disposal of This Product (Waste Electrical & Electronic Equipment)**

This marking shown on the product or its literature, indicates that it should not be disposed with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources.

Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this item for environmentally safe recycling.

Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial wastes for disposal.

Disposal of used batteries should be carried out in accordance with the national regulations for the disposal of batteries.

# 6. Some Useful Information

## 6.1 Temperature Measurements Taken within the Ear

This unit measures the temperature in the ear by sensing infrared emission of the eardrum and surrounding tissues, and then calculates the oral-equivalent value using a unique algorithm based on actual body temperature survey data.

The Gentle Temp 522 PRO is less threatening to a child than a rectal thermometer. It is faster, safer, and easier to use than an oral thermometer.

Clinical research has shown that the ear is an ideal site for taking body temperature.

## 6.2 Knowing a Patient's Normal Temperature

In order to make a correct judgment of suspected fever conditions, it is important to know the patient's normal temperature. Normal temperature varies with the individual and measurement time through a day. In addition, it may vary depending on the thermometer or the body part where you measure. To find out the patient's normal temperature, we recommend performing regular measurements under the following conditions:

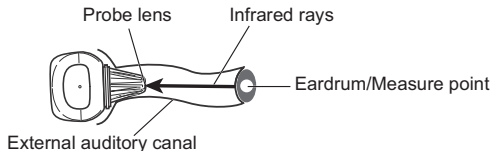
- Measure with the same thermometer.
- Measure at the same area.
- Measure at the same time of day.

## 6.3 Ear Temperature Compared to Other Types of Body Temperature

The normal temperature varies according to different locations on the body.

As ambient temperature, sweat, or saliva easily affects body temperature measured under the arm or the tongue, readings may be lower than the core temperature.

The temperature measured in the ear is different to that measured rectally. Tympanic temperature measurement accurately reflects the brain temperature and can lead to a quicker detection of fever.

One speaks of normal body temperature if the measurement value lies within a certain range. Body temperature varies however according to age.

Age	Normal ear temperature in °C and °F	
Babies	36.4°C - 37.5°C	97.5°F - 99.5°F
Children	36.1°C - 37.5°C	97°F - 99.5°F
Teens/Adults	35.9°C - 37.5°C	96.6°F - 99.5°F
Elderly	35.8°C - 37.5°C	96.4°F - 99.5°F

### 6.4 Questions and Answers

#### **How many times can I measure consecutively?**

You can measure consecutively up to three times. The main unit will then be warmed up and may not be able to measure correctly. If you are going to measure more than three times, wait for 10 minutes, then measure again.

#### **The temperature indicated is rather high.**

- 1 The probe cover may be faulty.
- 2 You may have used the thermometer that has been stored in a cool or cold place. Measure the temperature after leaving the unit in the room where you are going to use it for more than 30 minutes. If you store the unit in the room where you are going to measure the temperature, you can promptly use the thermometer.

#### **Is the temperature measured in the right ear different from that measured in the left ear?**

Among healthy people, there should be no significant difference in the measurement results. Differences may be caused by the following reasons:

- 1) The probe is not inserted in the same way.
- 2) Measurement is not conducted by inserting the unit in a stable manner in the same angle. Try to measure in the ear that consistently shows a higher measurement.

#### **The temperature shown is rather low.**

- 1 The probe cover is dirty.
- 2 The probe lens is dirty.
- 3 You removed the unit from the ear before the measurement is finished.
- 4 The ear is cold. The temperature tends to indicate low when someone uses an ice bag or an ice pack, or immediately after coming in from the cold in winter.
- 5 The thermometer is not inserted deep enough in the ear.

# 7. Optional Accessories

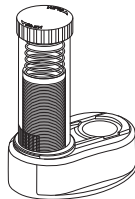
## Optional Medical Accessories

(within the scope of EC Medical Device Directive 93/42/EEC)

Probe Cover (MC-EP3-E)  
(200 pieces)

Probe Cover Loader (MC-EPL1-E)  
(Including 40 probe covers)

**Note:** Please check with your local OMRON representatives for appropriate optional accessories.

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