Easy Cuff HEM-RML31 (HEM-RML31-E)

• <i>Instructions for use</i>	2	EN
• <i>Mode d'emploi</i>	4	FR
• <i>Gebrauchsanweisung</i>	6	DE
• <i>Istruzioni per l'uso</i>	8	IT
• <i>Instrucciones de uso</i>	10	ES
• <i>Gebruiksaanwijzing</i>	12	NL
• <i>Инструкция для пользователя</i>	14	RU
• <i>إرشادات الاستخدام</i>	٢	AR

Easy Cuff HEM-RML31 (HEM-RML31-E)

Thank you for purchasing the OMRON Easy Cuff HEM-RML31. Arm circumference for this cuff is 22 to 42 cm. Please refer to the original instruction manual of your Blood Pressure Monitor.

 Please read this instruction manual thoroughly before using the unit. Please keep for future reference.

 **Caution:** Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

- Use only the approved OMRON Blood Pressure Monitor for this cuff.
- Use the same arm cuff as written on the side of the product by the air jack. Do not use the other types of the arm cuff.
- Do not wash the cuff or make it wet.

Instructions for use

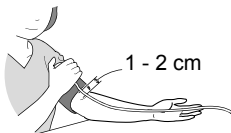
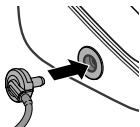
Remove tight-fitting clothing or tight rolled up sleeve from your left upper arm.

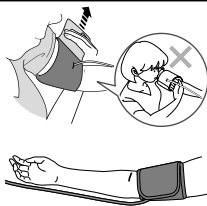
Do not place the arm cuff over thick clothes.

1. Insert the air plug into the air jack securely.

2. Wrap the arm cuff firmly in place around your left upper arm.

The bottom edge of the arm cuff should be 1 to 2 cm above the elbow. Air tube is on the inside of your arm and aligned with your middle finger.

3. Secure closed with the fabric fastener.

Taking measurements on the right arm

The air tube will be at the side of your elbow.
Be careful not to rest your arm on the air tube.

Note: The blood pressure can differ between the right arm and the left arm, and therefore also the measured blood pressure values can be different. OMRON recommends to always use the same arm for measurement. If the values between both arms differ substantially, please check with your physician which arm to use for your measurement.

- Use a soft and dry cloth, or a soft and moistened cloth and neutral soap to clean on the arm cuff.
- Do not use petrol, thinners or similar solvents to clean the arm cuff.
- Do not store the arm cuff to extreme temperatures, humidity, direct sunlight, dust or corrosive vapours.
- Do not bend or crease the air tube excessively.

Maintenance

This cuff fulfils the provisions of EC directive 93/42/EEC (Medical Device Directive) as well as EN1060 Non-invasive sphygmomanometers Part 1: General Requirements and Part 3: Supplementary requirements for electromechanical blood pressure measuring systems.

This OMRON product is produced under the strict quality system of OMRON HEALTHCARE Co., Ltd., JAPAN.

Product Description	Blood pressure monitor cuff
Model	Easy Cuff HEM-RML31 (HEM-RML31-E)
Arm Circumference	22 - 42 cm
Measurement Range	Pressure: 0 to 299 mmHg Pulse: 40 to 180 beats/min.
Accuracy	Pressure: ± 3 mmHg Pulse: $\pm 5\%$ of display reading
Service lifetime	1 year

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